

# RESMED

**Important:** Remove this guide before handing the device to the patient. Pressures should only be adjusted by qualified personnel.

## S8 Escape™

### QUICK SET-UP GUIDE



### Set-up is quick and easy . . . Just Go!







The following settings have been pre-set to accommodate typical patient treatment values.

#### S8 Escape™

Altitude:	0–2000 ft (0–609 m)
Ramp:	20 minutes
Max Ramp:	45 minutes
Start CPAP:	4 cm H <sub>2</sub> O
CPAP:	9 cm H <sub>2</sub> O
Reminders:	OFF







# 1

## Accessing the clinical menu and changing settings and options

- Turn on the device and wait until the standby (Ramp) screen appears on the LCD. At this stage you can adjust the ramp time.
- Press the **Down** and **Right** keys (  +  ) simultaneously for 3 seconds to access the clinical menu.
- Use the **Up**  and **Down**  keys to scroll through the menu.
- Use the **Left** key  to change each setting as required.
- Press the **Right** (exit) key  to return to the Patient Menu screen.

# 2

## Accessing and setting the reminder menu

- Turn on the device and wait until the standby (Ramp) screen appears on the LCD. At this stage you can adjust the ramp time.
- Press the **Left** and **Down** keys (  +  ) simultaneously for 3 seconds to access the reminder menu.
- Use the **Up**  and **Down**  keys to scroll through the menu.
- Use the **Left** key  to change each setting as required.
- Press the **Right** (exit) key  to return to the Patient Menu screen.

# S8 Escape menus

Important : retirez ce guide avant de remettre l'appareil au patient. Les pressions doivent être réglées uniquement par des personnels qualifiés.

## S8 Escape™

