


# REMstar

## M SERIES

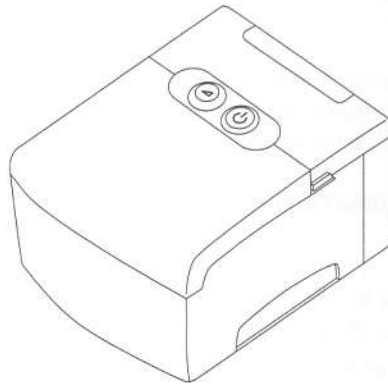

## Provider Quick Setup Guide




**IMPORTANT!** Remove this guide before giving the device to the patient. Only medical professionals should adjust device settings.

This guide provides you with basic instructions on how to navigate the provider menus to modify device settings. Refer to the *REMstar M Series Provider Manual* for more information.

### ACCESSING THE PROVIDER MODE SCREENS

---

Accessing provider mode unlocks additional settings that cannot be modified by the user. To access provider mode:

1. Plug the AC power cord into the power supply and an electrical outlet.
2. Hold down both the left and right   buttons while simultaneously plugging the power supply into the device. The Provider Mode Standby screen (shown on the next page) will appear.

You can view and modify prescription settings using the Provider screens.

**Note:** On all provider screens, the Unlock  symbol indicates that you are in Provider Mode.

**Note:** After 1 minute of button inactivity, the device times out and returns to the User Standby screen.

## PROVIDER SCREEN NAVIGATION

To navigate the display screens from the Provider Mode:



- Use the button to navigate to the next screen and the button to navigate to the previous screen.
- Use the and buttons to adjust the settings on each screen.

**Note:** When you have finished modifying or viewing settings, press the button to display the User Standby Screen.

**Note:** To use the device on a new patient, set the session counter (sessions >4) to zero. This also resets therapy hours and hours of patient use.

## ADDITIONAL SETTING INFORMATION

- **Altitude:**
  - 1 = Less than 2500 ft. / <762 m
  - 2 = 2500 to 5000 ft. / 763 to 1524 m
  - 3 = 5001 to 7500 ft. / 1525 to 2286 m
- **Fine Pressure Adjustment:**  
Allows you to calibrate the device so the pressure setting can be verified with a manometer. The range is -1.9 to +1.9 cm H<sub>2</sub>O.
- **Blower Hours:**
  - When this symbol appears in **Provider Mode** (with the unlock symbol ), it indicates total blower hours for this device.
  - When this symbol appears in **User Mode**, it indicates hours of patient use.
- **Compliance Check Value:**  
Allows you to verify reported patient compliance accuracy. Patients report their total therapy hours; total hours of patient use, total sessions >4, compliance check value, and the humidifier setting (if using one). When you enter the values into Encore® Pro, the software verifies patient compliance.

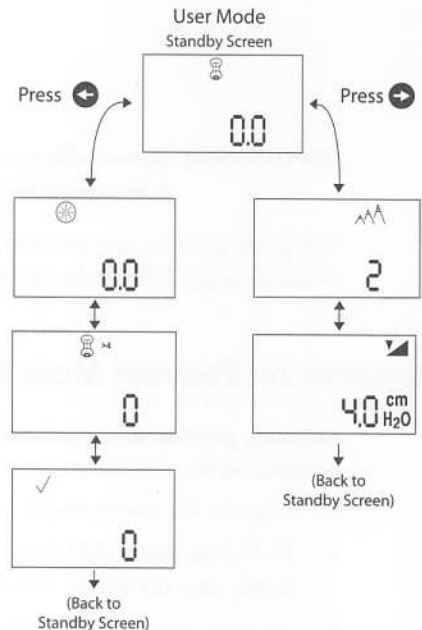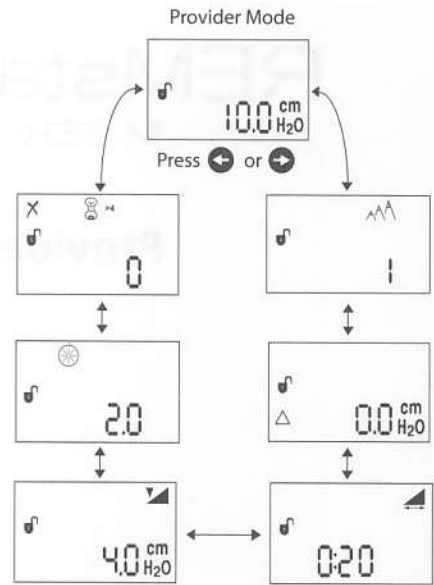

RESPIRONICS®  
1001 Murry Ridge Lane  
Murrysville, Pennsylvania  
15668-8550 USA

RESPIRONICS®  
Deutschland  
Gewerbestrasse 17  
82211 Herrsching Germany



0123

1030907  
AM 2/14/06